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### Indicações

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**Tissu 3D, Nylon, Polyester, Mousse PU, Cotton**

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## English Wrist Brace

### Indications

- To provide support for inflammatory conditions and soft tissue injuries, at the wrist (tendonitis, sprains and strains)
- Prevent further trauma caused by repetitive activity (overuse injuries)
- Secondary care after fracture treatment

### Contraindications

- Severe sprains
- Immediate treatment following fractures

### Precautions

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- The brace should fit snugly and be comfortable without interfering with circulation.
- Ensure appropriate wound care procedures are undertaken prior to application over open wounds or infected areas.

### Application

- Measure wrist circumference and take the appropriate size (for reference see size table instructions)
- Remove metal stay from the splint and adjust shape according to anatomy of the wrist or specific resting position requirements. Re-insert again in the pocket.
- Insert the straps in the D-rings and pull the brace over the hand with the thumb through the opening
- Attach the top and bottom straps to the hook, and wrap the center strap around the wrist with moderate tension
- A good positioned splint allows normal finger movement and is not too tight
- Cut off the excess strap length.

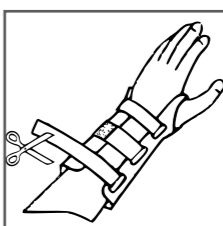
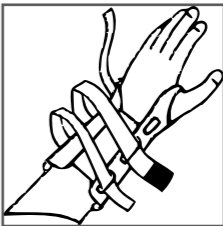
### Care & Cleaning

Remove aluminium stay from the pocket and fasten straps. Machine-wash at 30°C (delicate fabrics) or hand-wash in lukewarm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the brace is completely dry before re-applying it.

#### ⚠ Important information

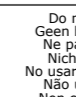
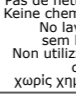
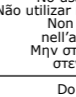
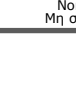
- This brace should only be applied in accordance with the instructions mentioned in this manual.
- The product is intended for single patient use.
- No liability is accepted for incorrect product use.
- Guarantee is bound to statutory regulations.

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