Vira x2

art no 50228

BINDER

2023-01-20

Size	Hip cm	Waist cm
XXS	65-80	50-65
XS	70-90	60-80
S	80-100	70-90
M	90-110	80-100
L	100-120	90-110
XL	110-130	100-120

Available colors:

STANDARD
100
OEKO
TEX®
1976-298
DTI

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MD Class I

This packaging can be recycled as paper and the bag as plastic.

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EN Elastic binder which provides efficient support over the abdomen. Easy to apply and adjustable to the desired fit and compression level with two drawstrings.

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EN Place the binder over the surged area. Begin by fastening the lower band **1** at the bottom of the hip. Then fasten the top band **2**. Adjust until it feels comfortable. The girdle should sit tight but not uncomfortably tight.

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EN Machine wash at 40°C (106°F) (can be washed at a higher temperature if necessary, the product will last longer if washed at a lower temperature). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN Postoperative girdle after abdominal/trunk surgery. Stabilizes in case of hernia/stoma. Preventive against hernia. **Contraindications:** Incarcerated hernia. Infection.

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EN Choose the size by the hip circumference, if the hip size is within an overlapping range, check the circumference of your waist:

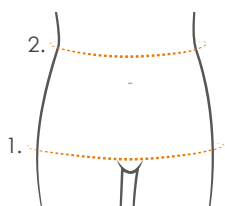
- Narrow waistline: choose the smaller size.
- Straight body shape: choose the larger size.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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Material

Polyester, 88%, Elastane 12%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.