INSTRUCTIONS FOR USE

SmartCuff - SHOULDER IMMOBILIZER

art no 20120

Package included

- A. Midjeband / Tailleband / Midjebånd / Vyötöröihina / Waistband / Taillenband
- B. Överarmsband / Overarmsbånd / Overarmsbånd / Olkavarsihihna / Upper arm cuff / Obere Armbinde
- C. Handledsbånd med vändsölor (C2+C3) / Händledsbånd med hægten (C2+C3) / Händledsbånd med regulator (C2+C3) / Rannehihna, jossa kiristyssoljet (C2+C3) / Wristband with D-ring (C2+C3) / Handschlaufe mit Spannschnallen (C2+C3)
- D. Axelband / Skulderstropp / Skulderstrapp / Olkahihna / Shoulder strap / Schulterriemen
- E. Positioneringskudde (används vid behov) / Positioneringspuude (anvendes om nødvendigt) / Posisjoneringspute (brukes om nødvendig) / Tukityyny (käytetään tarvittaessa) / Positioning cushion (used when necessary) / Lagerungskissen (wird bei Bedarf verwendet)

Application

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EN 1. Attach waistband **A** around the patient. Make sure velcro closure is positioned at front towards the injured arm. Cut **A** if too long. Position the injured arm to hang along the body.*

2. Fixate the arm in the upper arm cuff **B** and attach to **A**. Make sure **B** is closed tight, make sure **B** velcro closure is positioned at front to be easy for the patient to access. Cut off the excess material **B**.

3. Attach the wrist strap **C** wrist. Fix to **A**.

4. Attach the shoulder strap **D** to the waistband **A** at the back.

5. Pull the shoulder strap **D** over the shoulder and thread through the upper D-ring **C2** of the wristband **C**. Make sure length is correct, shorten the shoulder strap at the back if too long and fasten with Velcro.

6. Fixate the wrist in the wristband **C** by threading the outer band through the lower D-ring **C3**.

7. For more support of the upper arm place the positioning cushion **E** between the upper arm cuff and waistband.

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Material

Polyamide 67%, Polyurethane 12%, Polyester 21%. Latex free.

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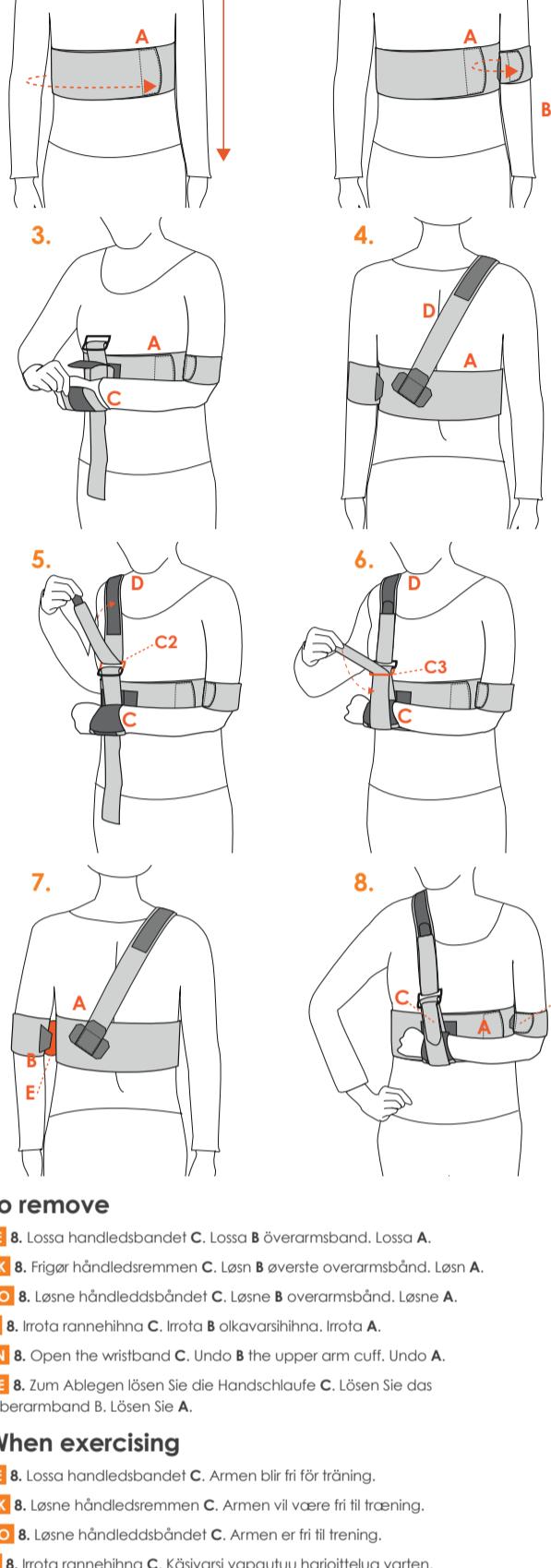
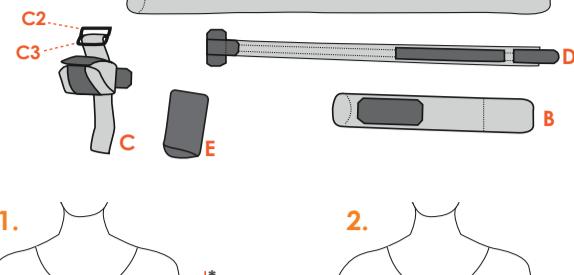
EN Machine wash at 40°C. Seal the Velcros. Use a laundry bag. Do not tumble dry. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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FI 8. Irrota rannehihna **C**. Irrota **B** olkavarsihihna. Irrota **A**.

EN 8. Open the wristband **C**. Undo **B** the upper arm cuff. Undo **A**.

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EN 8. Undo the wristband **C**. The arm is ready for exercise.

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Size	Circumference under breast	
One size	65 - 130 cm	

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

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NordiCare Ortopedi & Rehab AB