Kvick

art no 40300

WRIST BRACE

Measurement

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EN Indicated for Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains and post-cast removal.

Contraindications: There are no known contraindications or risks associated with correct use.

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EN Hand wash max 30° C. Dry flat. Seal the Velcros and remove the hinges before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

Material Composition

PU Foam 50%, Polyester 20%, Acrylic 15%, Nylon 15%. Spoon in aluminum. Latex free.

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EN Follow the instructions for use. If the product is prescribed by a medical professional, follow instructions as set out in the prescription throughout the rehabilitation period. Serious incidents shall be reported to NordiCare and the competent authority. Regularly check that the Velcro straps are free from dirt and that their fastening capabilities have not deteriorated.

Always contact your doctor/fitter: If you have questions, if you are taking medication, if you experience side effects such as skin irritation (remove the orthosis).