## Information

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. In case of pain, chafing or colour changes in the skin, remove the orthosis and contact the fitter.

## Material

49% Polyester, 25% Natural rubber latex, 14% Silicone, 12% Polyamid.

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**EN** Machine wash, 40°. Close the Velcro fastenings. Use a washing net. Do not bleach, do not dry clean. Do not iron or tumble dry.

Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20, info@nordicare.se.

**CE** This product meets the requirements of Medical Devices class 1, non sterile.

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**EN** 1. Place the belt over the pubis so that the cross-stitch and NordiCare label are in the middle front. The surface with silicone edges should be placed towards the body.

2. Place the straps behind your back with both hands. Switch the straps around behind your back.

3. Tighten the belt until desired support and fasten it with the Velcro straps at the front.

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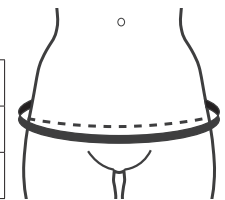
**EN** Counteract loosening of the joints/symphysisolysis during or after pregnancy. Also suitable as stable sacroiliac support for non-pregnant patients.

### Measurement

Size	Hip circumference
Small-Medium	75-110 cm
Large-X Large	110-145 cm

#### Tip if you want to shorten the belt:

If the belt is too long, you can cut at the marked seam along the edges of the strap. To prevent the material from fraying zig-zag the cut edge or carefully burn it with a flame to melt the material.