
LyftPlus Stomi

art no 50207-00

ABDOMINAL BINDER

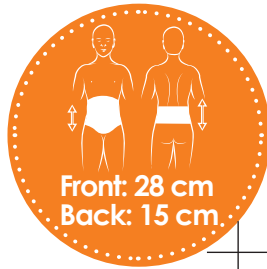2024-05-03

Available color:

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EN 1. Position the binder and secure the Velcro fastening at the side with the handle. If you have hernia, it may be preferable to put the binder on while lying down. This will help the hernia to sink in and ensure that the binder provides optimal support.

2. Adjust until comfortable. The binder should give a comfortable support, make sure it is not too tight.

3. Possible to cut hole for stoma in the front material.

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Material Composition

41% Polyamide, 39% Polyester, 15% Polyurethane, 5% Elastane. Latex free.

Size	Hip Circumference	Front height	Back height
M	90 - 105 cm	20 cm	15 cm
L	100 - 115 cm		
XL	110 - 125 cm		
2XL	120 - 135 cm		
3XL	130 - 145 cm		
4XL	140 - 155 cm		

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EN Mobilization after surgery. Stabilizing girdle for hernia/stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage.

Contraindication: Incarcerated hernia.

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EN Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN LyftPlus Stomi is 28 cm high at front and 15 cm high at back. For correct size measure the hip circumference in cm.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.