StomiFix

art no 50217

STOMA BINDER

2023-06-26
F3284.01

www.oeko-tex.com
6234-7001
DTI
STANDARD
100

Available colors:

Size	Hip Circumference
L/XL	120 - 140 cm
M/L	105 - 125 cm
S/M	85 - 110 cm
XS/S	65 - 90 cm

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EN 1. Put on the tubular shaped stoma binder just as you would with regular underwear. Make sure that the pouch hole is placed over the ostomy pouch. Pull the bag out through the hole.

2. Fold down the upper part of the binder to cover the bag. Secure the pouch in the folded edge at the bottom of the binder. Wear your regular underwear underneath or on top of the StomiFix.

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EN Machine wash in 60°C (140°F). The product lasts longer if washed in 40°C (104°F). Possible to tumble dry at low heat. Do not iron. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean.

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EN To be used for stoma to fixate/keep the ostomy pouch in place. **Contraindications:** Ostomy hernia.

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EN In order to determine the correct size, measure the hip circumference at its widest point.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

NordiCare

NordiCare Ortopedi & Rehab AB
Solrosvägen 1, SE-263 62 Viken, Sweden
Phone +46 42 35 22 20, Fax +46 42 35 22 21
info@nordicare.se, www.nordicare.se

MD Class I