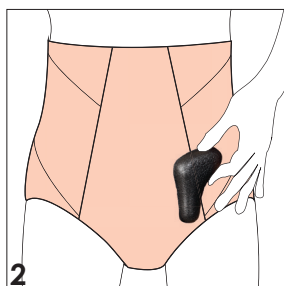
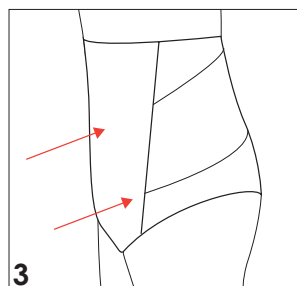
1

2

3

## Application

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**EN** To be worn instead of regular briefs. Flat pads (1) are included, convex pads (2) for increased pressure can be ordered separately. Provides lift and support (3).

- Place the pads in the pockets. Convex pads should be positioned with the convex part against the body. Remove pads before washing briefs.
- Put on the briefs first thing in the morning. Preferably laying down, when the hernia is most sunken in.
- Easily opened with hook and eye, when going to the toilet.
- To be worn directly against the skin. Do not use the briefs directly on damaged/irritated skin.
- Do not use with ointments or creams.

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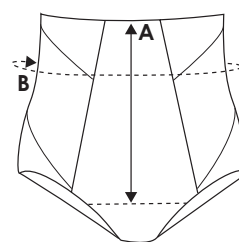
**EN** Waste circumference (B) in cm. Height is measured from the pubis and up (A).

Size	A	B
02	20	65-69
03	20	70-74
04	20	75-79
05	20	80-84
06	20	85-89
07	20	90-94
08	20	95-99

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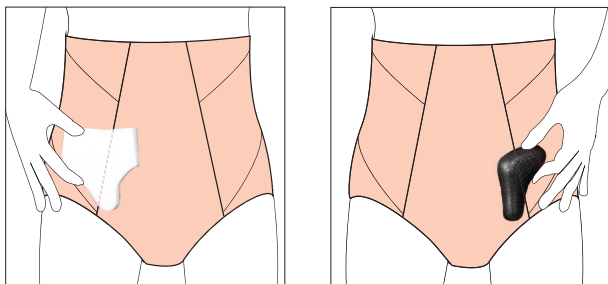
**EN** For conservative treatment of light/medium inguinal hernia and postoperatively to support the area of surgery and provide pain relief. Also after abdominal surgery/liposuction without pads.

**Contraindication:** Incarcerated hernia.

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result.

## Materials

Cotton 86%, Elastane 14%. Pads in Polyethylene.

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**EN** Machine wash at 40°C (104°F). Use washing bag. Iron in low temperature. Do not bleach. Do not dry clean. Do not tumble try.

For complete product information, see the data sheet or product catalog.