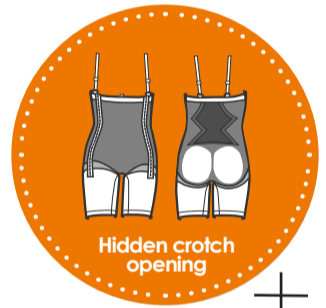
Compression Garments by NordiCare

Cindy

art no 51208

HIGH WAIST BODY

Compression class I (15-21 mmHg)

Available colors:

Compression class I (15-21 mmHg)

Adjustable with hook & eye

Seamless


4-way stretch

This packaging can be recycled as paper and the bag as plastic.

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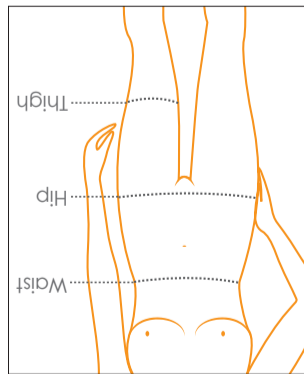EN Compression body with high waist in knitted seamless material. Recommended after surgery or liposuction on the abdomen, sides, back, seat and thigh. The body have a hidden opening in the crotch wedge, hook and eye closure in both sides and adjustable shoulder straps.

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EN 1. Make sure your skin is dry. Open the hook and eye closure.
2. Carefully pull the body up to the hip. Make sure that the crotch part is high/comfortable in the crotch and that the body is centered at the back.
3. Then pull the body over your abdomen and back, make sure to pull the back properly.
4. Close the hook/eyes in the side at the desired size/shape/compression. Make sure the flap behind the hook/eyes is flat against the skin, before you tighten the shoulder straps.

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EN Recommended after softer plastic surgery. Compression improves circulation and reduces the risk of post-operative swelling, leading to better healing and even pain relief. Corresponding compression class 1.

Contraindications: Infection. Arterial circulatory disorders. High risk of thrombosis.

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the product and contact your medical professional. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Serious incidents shall be reported to NordiCare and the competent authority.

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EN For best fit and correct compression it is important that measurements are taken carefully and size chosen according to the measuring table. The fit of the garment should be tight but comfortable. If the patients measurements differ from the measurement table, the garment should not be used. Keep in mind that measurements may vary during the healing process, a time after surgery when any swelling has gone down, a smaller size may be required.

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EN Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Do not tumble dry. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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