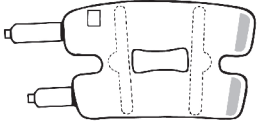
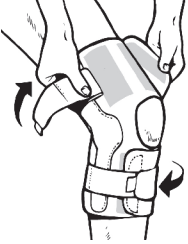
## HKW - Hinged Knee Wrap

<b>EN</b>	<b>KNEE ORTHOSIS</b>
<b>DE</b>	<b>KNIEORTHESE</b>
<b>FR</b>	<b>ORTHÈSE DU GENOU</b>
<b>ES</b>	<b>ORTESIS DE RODILLA</b>
<b>IT</b>	<b>ORTESI PER GINOCCHIO</b>
<b>NL</b>	<b>KNIEORTHESE</b>
<b>DK</b>	<b>KNÆORTOSE</b>
<b>FI</b>	<b>POLVIORTOOSI</b>
<b>SE</b>	<b>KNÅORTOS</b>
<b>NO</b>	<b>ORTOSE, KNE, ORDINÆR</b>
<b>PL</b>	<b>ORTEZA STAWU KOLANOWEGO</b>
<b>GR</b>	<b>ΝΑΡΘΗΚΑΣ ΓΟΝΑΤΟΣ</b>

<b>1</b>	
<b>2</b>	
<b>3</b>	

#### ENGLISH

##### Product Description

A short, functional soft knee wrap support with polycentric hinges that mimic the action of the knee.

##### Usage Information (See diagram)

- Open straps with tabs and place knee brace flat, “wrong” side up with the label showing.
- With the label in right hand, wrap top of knee brace around thigh and bottom of knee brace around calf. Close with initial contact closure to hold brace in place, then close all straps.
- Tighten all straps for comfort. Note: Posterior (rear) straps are used by therapist or orthotist to align hinges correctly. The patient should not alter these straps.

##### Indications for Use

- Mild instabilities of the knee.
- Meniscus injuries.
- Trauma or degenerative deformities of the knee joint.
- Protection in the later stages of rehab.

##### Contraindications

- Fractures of the proximal femur or distal tibia and fibula.
- Do not use directly on broken skin.

##### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Close over all hook/loop straps and remove metal hinges.
- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check Hook/loop fasteners are operating securely.

##### Product Family Composition

Aluminium, Elastane, Nylon, Polyamide, Polyester, SBR, Stainless Steel, Steel.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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