INSTRUCTIONS FOR USE

Marathon Tall - AIR WALKER

art no 80200

Indications

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EN Indicated for stable foot and ankle fractures, severe sprains, soft tissue injuries, post-operative and post-cast removal. **Contraindications:** No known.

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EN 40% Glass fiber reinforced polyamide, 25% Polyester, 15% Polypropylene, 10% Thermoplastic rubber (TPR), 5% TPU, 5% EVA. Latex free.

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EN Liner should be hand washed only using warm water and detergent for delicate fabrics, repeatedly rinsed and line dried only. Do not use bleach or place in a dryer. Wipe plastic shell with damp cloth, you may use disinfectant or mild soap and wipe dry. Periodically check the condition of the Velcro to ensure they are free from grease and damage.

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EN This product has been designed and tested based on single patient use only. It should never be used by multiple patients. If additional pain, swelling or symptoms occur while using the device, seek medical attention. Only medically trained persons are to prescribe and recommend the use of the product. Serious incidents shall be reported to NordiCare and the competent authority.

NordiCare®

Measurements & Size

SE Motsvarande skostorlek.

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DK Modsvarende skostørrelse.

EN Corresponding to shoe size.

NO Tilsvarende skostørrelse.

Shoe size	<35	36-38	39-42	43-45	>46
Size	XS	S	M	L	XL

Marathon Tall

art no 80200

AIR WALKER

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.

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Corflex, Inc
669 East Industrial Park Drive
Manchester, NH 03109,
www.corflex.com
Toll free: (800) 426-7353
Fax: 603 623 4111
CORFLEX
Designed for life.

MD Class I

This packaging can be recycled as paper and the bag as plastic.

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EN 1. Place leg inside liner and position heel as far back as possible. Close forefoot cover over top of foot and fasten it snugly **A**.

2. Wrap liner around the front of lower leg and secure with hook and loop closures **B**.

3. Situate front shell over liner taking care to properly position it over the inflation bulb **C**.

Check that the inflate/deflate knob **D** is in the DEFLATE position before tightening straps, this will allow air to escape from the bladder.

The front shell **E** should not contact the foot or leg, adjust as needed.

4. Insert ankle strap **F** through plastic buckle and secure strap. Always secure this strap first to encourage proper positioning in the walker.

5. Secure forefoot strap **G**. Insert secondary straps **H** through buckles and fasten back on itself.

6. Test for desired comfort level, do not overtighten straps.

CAUTION: DO NOT OVERTIGHTEN STRAPS, TIGHTEN STRAPS UNTIL SECURE AND COMFORTABLE

7a. To inflate liner, turn the inflate/deflate knob **D** clockwise until fully opened and press and release inflation bulb **C** until liner is providing support and comfort.

INFLATE LINER CAUTION: DO NOT OVERINFLATE LINER

7b. To release air from liner, turn the inflate/deflate knob **D** counterclockwise several times. If at any time the liner feels too tight, immediately release air by turning the inflate/deflate knob counterclockwise.

I: CLOCKWISE INFLATES LINER

J: COUNTERCLOCKWISE RELEASES AIR FROM THE LINER

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- DEFLATE LINER BEFORE REMOVING THE AIR WALKER.
- PRESSURE CAN BE ADJUSTED BY TIGHTENING AND LOOSENING STRAPS.
- DO NOT OVERINFLATE.
- FOLLOW PHYSICIANS INSTRUCTIONS.

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Troubleshooting:

If you cannot inflate or deflate the liner check that the metal valve located on the top of the inflation bulb is not blocked.

If you cannot inflate or deflate the liner check that the inflate/deflate valve is in the correct position.