# INFORMATION FOR WalkerROM - ANKLE ORTHOSIS

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**EN** The shell is cleaned with a damp cloth. The padding should be hand washed max 30° and dried flat. Do not tumble dry.

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**EN** Always read all instructions carefully before using this product. When in doubt, consult your doctor/device fitter. It is important that you follow the general and individual user instructions you receive from your doctor/device fitter throughout your rehabilitation period. Prescription, initial fitting and recommendation on how to use this product should be made by medically trained person. If changes in color of skin or other blood circulation related problems and skin irritation occurs, immediately remove the brace. The brace is only for one patient use. Serious incidents shall be reported to NordiCare and the competent authority.

## Material Composition

32% Polyamide,  
39% PU-foam,  
29% Polyester.  
Latex-free.

For more information, see productleaflet or [www.nordicare.se](http://www.nordicare.se). You can also contact your fitter or NordiCare: +46 42-35 22 20 / DK 7070-1907, [info@nordicare.se](mailto:info@nordicare.se).

**MD** Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

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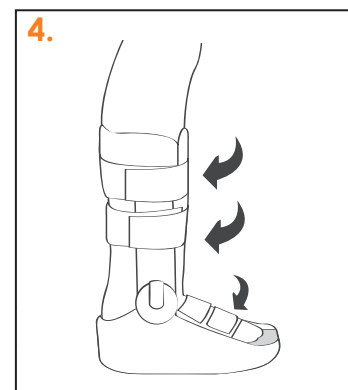
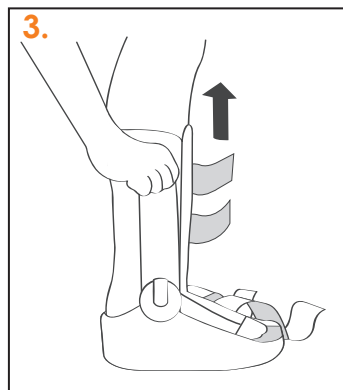
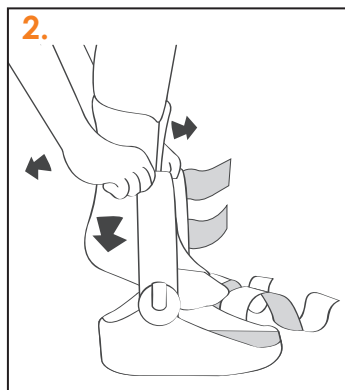
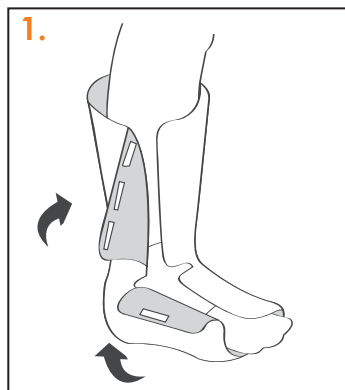
**EN 1.** When applying walker for first time open all straps and remove liner. Apply liner ensuring the heel is pushed fully into the liner. Fasten hook closures on foot and leg sections ensuring they are snug.

**2.** Pull uprights apart and place foot/liner into walker ensuring heel is against posterior border of walker. Uprights should be aligned with mid ankle and lower leg. (Metallic uprights are possible to bend to follow the contour of the leg.)

**3.** Apply pressure to ensure uprights are fixed to liner.

**4.** Fasten straps from toes up. Ensure they are firm but not too tight.

**5.** Set the desired range of motion for the hinge. Can be adjusted from neutral to 45° flexion and 45° extension, in intervals of 7,5°. Can be locked in fixed position, set for limited range of motion or set for free movement. Ensure that both hinges have the same setting.

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**EN** For increased comfort, socks in 2-pack are sold separately. Hygiene protection sold in 10-pack.

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